# BÉABA®

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**BÉABA®**

## MEASUREMENT GUIDE TO CHOOSE YOUR CORRECT NIPPLE SHIELD SIZE (OR FLANGE INSERT)

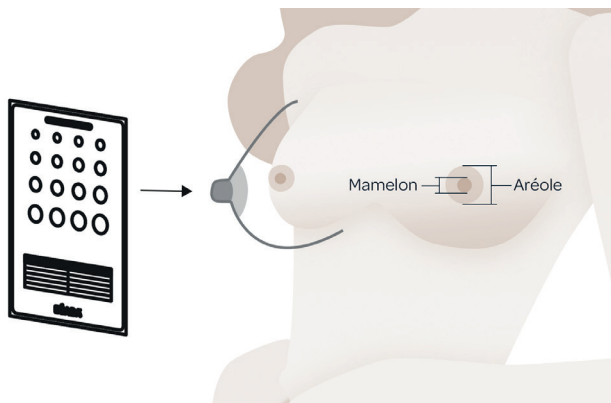
It's important to use a nipple shield that fits your size. This can not only increase your milk production but also make expressing more comfortable.

### HOW TO USE THIS GUIDE

1.  Print the following page
2. After printing, check with a ruler that your document is at the correct scale

3. Fold the sheet along one of the grey lines; this will make it easier to cut the circles.  Then, cut the half-circles along the dotted lines. When you unfold the sheet, you will be able to insert your nipple into the round openings.
4. Nipple measurement should be done before pumping. Relax to promote an accurate measurement. Remove any clothing or bra that could interfere with the measurement. It is not necessary to stimulate the nipple before measurement. The natural size of the nipple at rest is the one you should measure.

#### 5. Nipple measurement

Align an opening on the size guide with the base of your nipple. Do not include the areola area. Make sure your nipple fits comfortably into the opening without rubbing or compressing. Try different openings until you find the one that best fits you.

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Read the size indicated on the guide that corresponds to your nipple. Refer to the size chart to select the recommended nipple shield / flange insert size based on your nipple diameter.

Why the size of your nipple is not the same as the flange size:

To determine the correct flange size, you need to add a few millimeters to your nipple measurement. This extra space allows your nipple to expand comfortably during milk expression. For example, if the base of your nipple measures 21 mm in diameter, the ideal flange size for you would be 25 mm.

#### 7. Verification

Once you've selected a nipple shield size, try it on to ensure it's comfortable. The flange should surround your nipple without compressing the areola. Your nipple should move freely and comfortably during pumping. If you experience discomfort or pain during use, try a different size. It's normal for your nipple shield size needs to change over time, so don't hesitate to remeasure if necessary.

## NIPPLE MEASUREMENT

11 mm

12 mm

13 mm

14 mm

15 mm

16 mm

17 mm

18 mm

19 mm

20 mm

21 mm

22 mm

23 mm





24 mm

25 mm

26 mm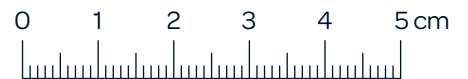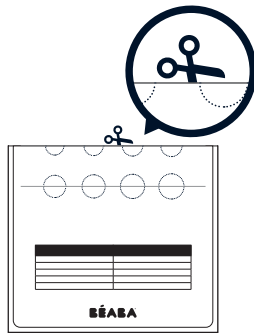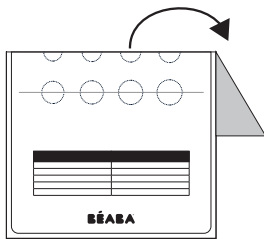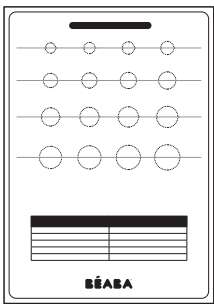

Nipple diameter (mm)	Recommended nipple shield or flange insert size (mm)
11 - 13 mm	15 mm
13 - 15 mm	17 mm
15 - 19 mm	21 mm
19 - 23 mm	25 mm
23 - 27 mm	29 mm

# BÉABA®