BÉABA®

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EN

WELCOME

Thank you for choosing our Béaba double electric breast pump to help you with breastfeeding. Breast milk is the most natural food for your baby which is why Béaba has designed an efficient and comfortable breast pump to help breastfeeding mothers on this great adventure. Before starting, please read these instructions carefully to ensure that you use our product under the very best conditions and keep it for as long as possible.

WARNINGS AND SAFETY PRECAUTIONS

Read and keep these instructions. Please take account of all the warnings.

- Only use the Béaba electric breast pump for the purpose described in this manual.
- Do not use the electric breast pump while driving, sleeping or if you are feeling excessively drowsy.
- This breast pump is intended for repeated use by **one sole person**.
- This appliance can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- Expressing your milk can trigger contractions. Do not express your milk before giving birth. If you fall pregnant and you are breastfeeding or still expressing your milk, consult a healthcare provider before continuing.
- Wash all the parts that come into contact with the breast and breast milk before using for the first time and after each use. Wash your hands carefully before handling the parts and the breast pump to avoid contamination. Refer to the “Cleaning” section for further information.
- Supervision is required when the Béaba electric breast pump is used near children.
- Children must not play with the appliance.
- The appliance must not be cleaned or maintained by children without supervision.
- Do not use the electric breast pump or one of its components if they are damaged.
- Do not place the electric breast pump near appliances that generate a strong electromagnetic field.
- It is advised to consult an approved healthcare professional before using this breast pump if you have an active implanted device that might be affected by a magnetic field (for example, a pacemaker or an implantable cardioverter-defibrillator).
- If expressing causes discomfort or pain, stop the appliance and place your finger between the breast and the nipple shield to separate them. Then, remove the D19 shield from the breast.

- Expressing your milk should not cause discomfort or pain; if you are unable to express your milk after 5 minutes of use, stop your session and try again a little later. **Consult a healthcare professional if you have a doubt.**

Warnings to avoid the transmission of infectious diseases:

- If you are a mother and infected with hepatitis B, hepatitis C or HIV (human immunodeficiency virus), expressing breast milk will not reduce or eliminate the risk of transmitting the virus to your baby through breast milk.

Precautions when using the electrical unit/lithium battery

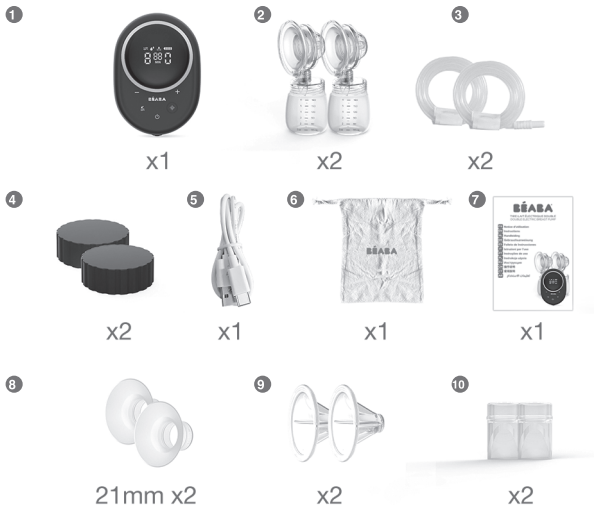
- The lithium battery is built-in and cannot be removed. Do not dismantle.
- Do not expose the electrical unit to direct sunlight or heat.
- Recharge the breast pump once a month when not used regularly.
- Repairs must be carried out by Beaba: contact our customer service.
- Do not leave the electric breast pump unsupervised when connected to a power source.
- Unplug the mains adapter from the wall socket when the breast pump is not in use, unless the battery is charging. Unplug the mains adapter when charging is complete.
- Keep the electrical unit dry and do not immerse it in water or place it under running water.
- Do not pick up an electrical appliance that has fallen into water. Unplug it immediately.
- This breast pump is protected against overheating: when the normal operating temperatures are exceeded, the pump unit triggers an alarm, switches off and stop charging.
- This appliance must only be supplied at safety extra low voltage corresponding to the marking on the appliance.

THIS APPLIANCE IS NOT A MEDICAL DEVICE.

INTENDED USE

The Béaba electric breast pump is designed to express the breast milk of breastfeeding mothers. The breast pump can be used in the event of difficulties encountered with breastfeeding or when mother and baby are separated. The appliance is designed to be used by just one person.

OVERVIEW OF PRODUCT & ACCESSORIES

- 1 Pump electrical unit
- 2 Pre-assembled 180 ml collection bottles (PP: polypropylene, x2), 25 mm nipple shields (silicone, X2), pump bodies (PP: polypropylene, x2), diaphragms (silicone, x2), backflow valve (silicone x2), hygiene covers (PP: polypropylene, x2)
- 3 Tubing with connectors (silicone / PP: polypropylene, x2)
- 4 Bottle lids (PP: polypropylene, x2)
- 5 USB/ USB-C cable
- 6 Storage bag (cotton, x1)
- 7 Instructions
- 8 21 mm flange inserts (silicone, x2)
- 9 Additional diaphragms (silicone, x2)
- 10 Additional backflow valves (silicone, x2)

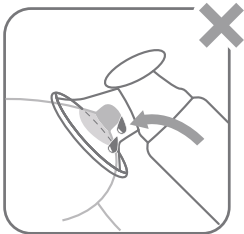
FUNCTIONS

This breast pump offers the following functions and characteristics:


- Milk collection kit with silicone nipple shields for gentle pressure,

vacuum and massage to stimulate lactation while expressing milk.

- Easy assembly and cleaning.
- Expressing in three phases that imitates baby's natural suckling.
 - "Stimulation" mode: effectively massages the nipple and areola to stimulate the ejection reflex. 9 adjustable power levels.
 - "Express" mode: imitates baby's suckling rhythm. 9 adjustable power levels.
 - "Turbo" mode: Helps to extract more milk and empty the breast. 9 adjustable power levels.
- "Memory" function to automatically save the settings for each mode from the previous time used for the next time.
- Equipped with a «comfort» expressing kit. Unlike traditional breast pumps, our breast pump is designed to enable comfortable expressing in a semi-recumbent position. Thanks to the optimised angle between the nipple shield and the bottle, the milk can flow naturally without any risk of backflow, thus offering unequalled comfort to mothers who prefer to rest while expressing.

Classic breast pump

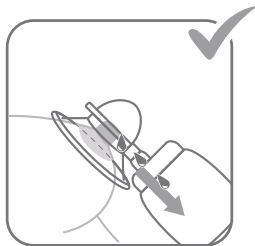**Breast pump with
«comfort expression
kit»**

- "Night light" function: handy for night use.
- Backflow design: silicone membrane acting as a barrier to prevent the backflow of breast milk.
- Low noise level for a quiet and comfortable experience.
- Rechargeable with a mains adapter. Built-in rechargeable lithium-ion battery.
- USB-C connection, recommended mains adapter 5VDC – 2.0A 10W (EC)

SELECTING THE CORRECT NIPPLE SHIELD SIZE

- 1) Measure the diameter of the base of your nipple (in mm).
- 2) Add 4 mm to this measurement, this is the size of your nipple shield. A space of about 4 mm between your nipple and the nipple shield allows the nipple to expand while pumping your milk. For example, if the base of your nipple measures 21 mm in diameter, the size of your nipple shield should be 25 mm.

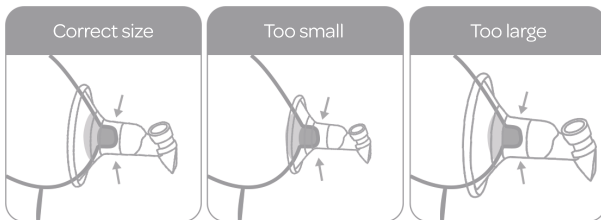
N.B.: your breast pump is supplied with 25 mm nipple shields and two 21 mm flange inserts (which are the most commonly used sizes). However, if necessary, you can order 15, 17 or 19 mm flange inserts from our customer service.

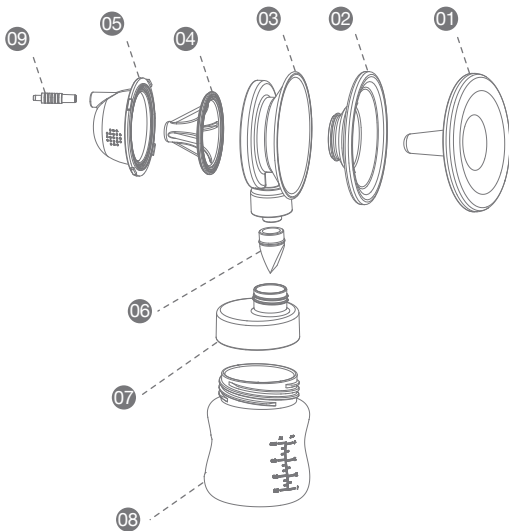
If you are uncertain about the size of the nipple shield you require, print our measurement guide by scanning the QR code below.

How to know whether the nipple shield is the right size?

- Before pumping your milk, it should be possible to insert your nipple with very little or no areola in the tunnel.
- While expressing your milk, your nipple should be centred in the middle and be entered and removed from the tunnel without rubbing, redness or pain.


PREPARING THE BREAST PUMP

- ❶ Hygiene cover (PP: polypropylene, x2)
- ❷ 25 mm nipple shield (silicone, x2)
- ❸ Body of breast guard (PP: polypropylene, x2)
- ❹ Diaphragm (silicone, x2)
- ❺ Diaphragm cover (PP: polypropylene, x2)
- ❻ Backflow valve (silicone, x2)
- ❼ Bottle lid (PP: polypropylene, x2)
- ❽ 180 ml bottle (PP: polypropylene, x2)
- ❾ Tubing connector (PP: polypropylene, x2)
(+ 2 21mm silicone flange inserts)

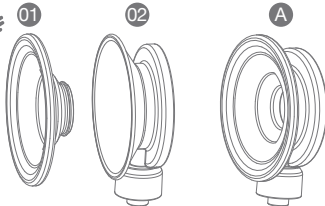
1 - Assembling the nipple shield


Place the breast cushion (nipple shield) 01 in the body of the nipple shield 02 to obtain the complete kit A. Make sure that both sides of the silicone cushion cover the sides of the pump body.

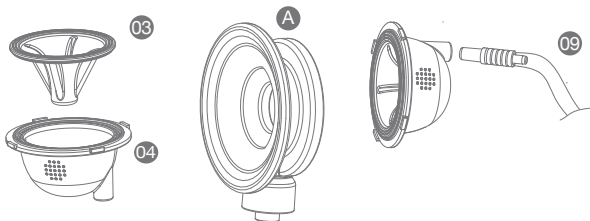2 - Assembling the diaphragm

Place the diaphragm 03 in the diaphragm lid 04 and screw it into the complete Kit A. Then, insert the tubing connector 09 into the diaphragm lid 04. Make sure that they are correctly sealed to avoid leaks.

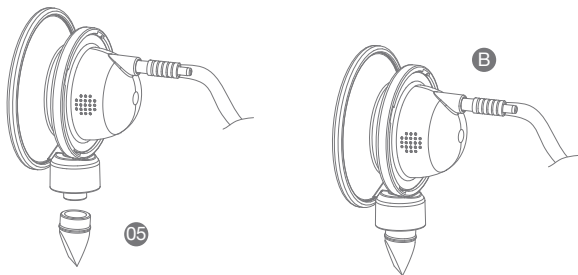3 - Assembling the valve


Insert the valve **05** firmly into the pump body outlet. You now have a complete Kit **B**.

Avoid rubbing the pump body outlet too hard in the backflow valve and replace the valve if it breaks, otherwise the suction function will be damaged.


The backflow valve is a fragile component; avoid damaging it when cleaning. Your Béaba breast pump is supplied with spare backflow valves. Contact our customer service if you require additional parts.

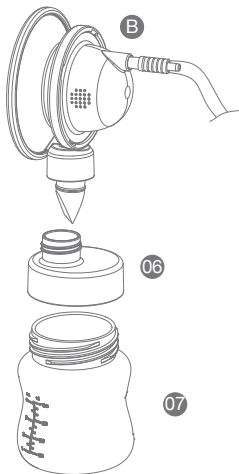4 – Assembling the Bottle


Screw the bottle connector **06** firmly onto the complete kit **B** and the bottle **07**. Your first milk collection kit is ready. Repeat the same procedure for the second kit.

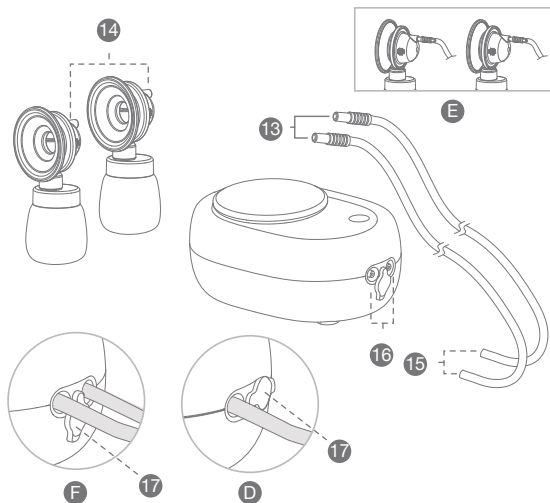5 - Connecting the pump motor base to the milk collection kits
Insert the tubing connectors **13** to the two ends **14** of the diaphragm covers. Insert the other end of each tubing **15** into the pump unit **16**. Make sure that the pump unit and milk connection kit are properly connected **F**.


Comment: When it is necessary to convert the double breast pump (configuration **E**) into a single breast pump (configuration **D**), simply remove one tubing from the pump unit and close the remaining air vent with the cap supplied (**17**). Caution: Switching from double pumping to single pumping will increase the breast pump's power. Reduce the power level beforehand to avoid starting at too high a power which could be uncomfortable. Check that cap **17** is correctly in place to ensure a good seal.

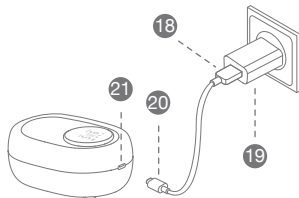6 - Connecting the pump unit to the power supply for recharging
 Insert the cable's USB-C port **18** into your mains adapter **19**. Then insert the cable's USB-C **20** connector into the port on the pump unit **21**. Connect the output of your mains adapter **19** to the power socket and then plug the adapter into the mains. It takes between 1 and 2 hours to fully charge your appliance.
 Recommended mains adapter 5VDC – 2.0A 10W (EC).
 Using an unsuitable mains adapter will invalidate the warranty.

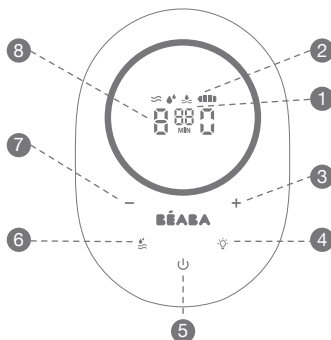DISPLAY SCREEN AND COMMANDS

- 1 Timer: duration of the expressing session in minutes
After starting the pump unit, the display will count from 0 minutes to 30 minutes and indicate the duration of the session. After 30 minutes, the pump unit switches off automatically and enters standby.
- 2 Battery level: when the battery is almost empty, the symbol flashes on the screen.
- 3 Increase power.
- 4 Night light mode. In standby or operating mode, touch the button to switch on the night light.
- 5 On/off or pause/resume button (long press for on/off, short press for pause/resume).
- 6 Mode selection:
 - a. Stimulation mode \approx : 9 power levels
 - b. Express mode \bullet : 9 power levels

- c. Turbo mode  : 9 power levels
- After switching on the unit, press and hold the button to start pumping your milk. By default, the breast pump will enter “stimulation” mode. Press the mode button  to switch the breast pump to express/turbo mode.

-  Reduce power.
-  Suction level.

Comments:

- the breast pump has a “memory” function to automatically save the power setting for each mode from the previous time used for the next time.
- “silence” function: press and hold the “-” “+” buttons to mute the sound of the buttons.

LITHIUM BATTERY POWER SUPPLY/CHARGING

- In operating mode, the pump unit is powered when the mains adapter is connected; when the mains adapter is not connected, the pump unit automatically uses the built-in lithium battery for power.
- The lithium battery must be charged fully for about 3 hours. Once fully charged, it can work continuously for about 2 hours in pumping mode on both sides.
- When the battery level is too low, the pump unit stops working automatically and an alarm is emitted to protect the lithium battery. When the battery is full, it stops charging automatically to avoid damaging the lithium battery.

INSTRUCTIONS

- 1) Clean well and check the condition of your appliance and the milk collection kit (see following section).
- 2) Switch on the appliance . Your appliance is on but in sleep mode.
- 3) Press the button  again to start pumping your milk. By default, the breast pump will enter “stimulation” mode. The breast pump will start stimulating the milk ejection reflex with 9 adjustable power levels.
- 4) Press the “-” or “+” button ( or ) to increase/decrease the vacuum in stimulation mode. Select one of the 9 levels. The display will indicate the power level. The appliance switches automatically to express mode after 3 minutes of stimulation. You can ignore this 3-minute period by selecting express or turbo modes.
- 5) When the milk ejection reflex commences or when the milk starts to flow:
 - a. Press the button  to commence express or turbo mode.
 - b. Use the “-” or “+” buttons ( or ) to adjust the power

level to a comfortable setting to start expressing your milk. Select one of the 9 levels. The display will indicate the power level.

- c. You can pause and then start expressing your milk again by briefly pressing the on/off button 5. If you pause for more than 2 minutes, we recommend restarting stimulation mode.

- 6) At the end of your milk collection session, switch off the electrical unit by pressing the on/off button 5 for a few seconds.
- 7) Then, unscrew the bottle(s) and close with the lids. The two elements (bottle and lid) can be used in the fridge and freezer. You can store your expressed milk in bottles or pour it into a recipient of your choice suitable for storing milk.

8) **Recommendations for storing your milk in the bottles supplied:**

WARNING!

- Always remove the bottle from the motor unit before storing.
- Always make sure that the bottles and their lids are not cracked or damaged before each use.
- The bottle must not be used to reheat breast milk. Do not use a microwave or bain-marie to reheat milk.
- Do not defrost the milk in the open but in the fridge.
- Wash your hands carefully before touching the bottles and avoid touching the inside.
- Never refreeze defrosted breast milk.
- Always check the temperature of the milk before serving it to baby.

Recommendations:

- Remove the bottle from the electrical unit after expressing your milk and close it with the lid supplied.
- Remember to label the bottle with the date and time the milk was expressed and the volume of milk collected.
- Be sure not to fill the bottle to the maximum limit (freezing will increase the volume of milk in the bottle).

Breast milk can be refrigerated, frozen and defrosted in the BEABA bottle. Please follow the instructions below:

Breast milk	Storage	Shelf life
Fresh	Room temperature (22°C max)	4 to 6 hours
	Fridge (0 to 4°C)	3 to 5 days
	Freezer (freezer compartment of fridge)	2 weeks
	Freezer (-18°C)	6 to 9 months
Defrosted	Room temperature (22°C max)	2 hours
	Fridge (0 to 4°C)	24 hours

- **Do not refreeze after defrosting.**
 - **Please note that these times cannot be combined.**
 - These storage and defrosting rules for breast milk are recommendations only.
 - **Consult a healthcare professional or a breastfeeding specialist for further information.**
- Defrosting milk:
- Do not use a microwave or bain-marie to defrost breast milk.
 - Defrost the milk gradually in the fridge.
 - In the event of an emergency, you can defrost breast milk by placing the bottle in cold or lukewarm water (max. 37°C) and using it immediately.
 - To reheat the milk, pour the milk into a suitable container and refer to the manufacturer's instructions.

CLEANING YOUR MILK COLLECTION KIT

Recommendations:

- **When using for the first time**, dismantle, wash and sterilise all the parts of the breast pump that come into contact with the breasts or breast milk. **After each use**, dismantle and wash all the parts of the breast pump that come into contact with the breasts and the breast milk.
- Dismantle and clean the parts that come into contact with the breasts and the breast milk immediately after use to avoid the build-up of milk residue that might favour the growth of bacteria.
- Take care when cleaning the silicone cushion (nipple shield, diaphragm and valve). Your breast pump will not work correctly if they are damaged.
- Always wash your hands before touching the breast pump and its parts. Avoid touching the internal parts of the milk collection kit that come into contact with breast milk.
- The electrical unit must **never** be exposed to or immersed in water or other liquids to clean it. Never wash and never sterilise the pump's motor, this part does not come into contact with breast milk. The electrical unit's motor contains electrical components that must remain dry. A clean, damp cloth can be used to clean the pump's motor. Any other cleaning method is considered to be incorrect use and will cancel the warranty.
- The appliance **MUST NEVER BE USED** without a diaphragm: risk of fluids entering the appliance's electrical part.
- **NEVER USE** the product without the backflow valve: suction will be irregular and the appliance will be inefficient.
- Always check the tubing and all the plastic or silicone parts for wear.
- Replace used or damaged parts immediately because they will affect the operation of your breast pump.
- **NEVER RINSE OR PLACE THE TUBING UNDER WATER: the narrow diameter makes drying very difficult.**
- The breast pump's milk collection kit has a closed system, therefore it is not necessary to clean the tubing.
- If you notice humidity in the tubing after the milk collection session, remove the tubing from the nipple shields. Switch on the pump unit for a few minutes until the humidity disappears.

Washable components:

- | | |
|--------------------------------------|-----------------------------------|
| 1) Diaphragm (silicone) | 5) Bottle lid (PP: polypropylene) |
| 2) Pump body (PP: polypropylene) | 6) 21 mm flange insert (silicone) |
| 3) Backflow valve (silicone) | 7) 25 mm nipple shield (silicone) |
| 4) 180 ml bottle (PP: polypropylene) | |

Always handle these elements gently when cleaning. Your breast pump will not work correctly if they are damaged.

• **Cleaning the washable components (before using for the first time and immediately after use)**

- 1) Rinse the elements in cold, clear water (about +20°C / +68°F).
- 2) Wash the elements in lukewarm water (about +30°C / +86°F) with a fragrance-free, liquid washing-up product for babies.
- 3) Rinse in cold, clear water (about +20°C / +68°F) for 10 to 15 seconds.
- 4) Place all the cleaned parts on a paper towel or on drip tray and leave to dry. Avoid using dish cloths because they may contain germs and bacteria that might contaminate the breast milk.
- 5) Make sure that all the parts are completely dry before use. Once all the parts are dry, reassemble the milk collection kit.

OR

- 1) Place the elements on the top rack of the dishwasher.
- 2) Make sure that they are completely dry. If necessary, place all the cleaned parts on a paper towel or on drip tray and leave to dry. Avoid using dish cloths because they may contain germs and bacteria that might contaminate the breast milk.
- 3) Make sure that all the parts are completely dry before use. Once all the parts are dry, reassemble the milk collection kit.

• **Sterilising the washable components before using for the first time**

- 1a) Boil in water for a maximum of five minutes.
 - a. During sterilisation, use a teaspoon of citric acid or a little vinegar in 200 ml of water to descale and degrease the pump's parts. Limescale and grease will affect the vacuum and the transfer of milk to the collection kit.

- b. Boiling or exposing to heat too often may cause wear of plastic and silicone parts, which will affect the operation of your breast pump.

OR

- 1b) Use steam or cold water sterilisation methods. We do not recommend sterilising in a microwave.
- 2) Place all the cleaned parts on a paper towel or on drip tray and leave to dry. Avoid using dish cloths because they may contain germs and bacteria that might contaminate the breast milk.
- 3) Make sure that all the parts are completely dry before use. Once all the parts are dry, reassemble the milk collection kit.

• **Cleaning the electrical unit:**

Disinfect the electrical unit with disinfectant wipes. Wipe the unit with a clean, damp cloth or paper towel before use. Never place the electrical unit under water and never immerse it. Do not use solvents or abrasive products to clean the electrical unit.

Scan the QR code below to watch our explanatory video:

TROUBLESHOOTING

Low suction/no suction

CHECKS	CORRESPONDING ACTION
Check whether the silicone nipple shield is deformed or stained.	The deformed silicone cushion must be replaced, reinstall it if it does not fit properly, the stains must be cleaned.
Check whether the diaphragm is installed upside down or damaged.	Reinstall or replace the diaphragm. Check that it is positioned correctly and that the diaphragm's cover is correctly in place.
Check all the connections and check that the diaphragm cover is tightened, if the tubing connector and the tubing and diaphragm cover are inserted correctly.	Tighten the connections.
Check whether the tubing connector is inserted too far into the machine.	When the tubing connector is inserted in the machine, do not force too hard on the connection. It will not fall out after being inserted gently.
Check whether there is steam or a backflow of milk in the tubing.	Clean the tubing, shake out the water droplets and dry.
Check whether the tubing is knotted, folded or pinched.	Unfold the tubing while in use, check that no part is pinched.
Check that the lid is connected correctly to the second vent when using just one bottle.	Replace the lid on the second air vent when using just one bottle.

Painful, inefficient use, irregular suction

Check that the nipple shield used suits your body shape.

See section “Selecting the correct shield size”

Backflow valve damaged or incorrectly installed.

Check the condition of the backflow valve. Replace it if it is damaged.

Pain when using: power level too high.

Reduce the power.

Does not charge or switch on

Check that the adapter model is correct and that the adapter is plugged in.

Check the adapter model and insert it again.

Check that the battery is not out of charge.

Try again after charging the battery.

Check whether the adapter is faulty.

If, no charge is displayed on the screen after plugging in the adapter, the adapter is faulty.

If your problems with the Béaba breast pump persist or if you have any other questions, please contact our consumer service.

TECHNICAL DATA

Recommended mains adapter: Input 100-240V 50/60Hz, Output 5VDC, 2A – 10W

Battery type and capacity: Li-ion 3.7V 2200mAh

Timer: 30 minutes

Noise level: ≤ 45 dB

Protection class: class III

Operating conditions: temperature 5-40°C, relative humidity 10%-90%

This symbol, displayed on the product or on its packaging, indicates that this product complies with the European Directive 2012/19/EU and must not be disposed of with household waste. It must be taken to an appropriate collection point for recycling electrical and electronic equipment. By ensuring that this product is disposed of in an appropriate way, you will be helping to prevent potential harmful consequences for the environment and human health. Recycling materials will help to conserve natural resources.

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BEABA ASIA LTD