# BÉABA®

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**EN**

## WELCOME

Thank you for choosing our Béaba single electric breast pump to help you with breastfeeding. Breast milk is the most natural food for your baby which is why Béaba has designed an efficient and comfortable breast pump to help breastfeeding mothers on this great adventure. Before starting, please read these instructions carefully to ensure that you use our product under the very best conditions and keep it for as long as possible.

## WARNINGS AND SAFETY PRECAUTIONS

Read and keep these instructions. Please take account of all the warnings.

- Only use the Béaba electric breast pump for the purpose described in this manual.
- Do not use the electric breast pump while driving, sleeping or if you are feeling excessively drowsy.
- This breast pump is intended for repeated use by one sole person.
- This appliance can be used by persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved
- Expressing your milk can trigger contractions. Do not express your milk before giving birth. If you fall pregnant and you are breastfeeding or still expressing your milk, consult a healthcare provider before continuing.
- Wash all the parts that come into contact with the breast and breast milk before using for the first time and after each use. Wash your hands carefully before handling the parts and the breast pump to avoid contamination. Refer to the “Cleaning” section for further information.
- Supervision is required when the Béaba electric breast pump is used near children.
- Children must not play with the appliance.
- The appliance must not be cleaned or maintained by children without supervision
- Do not use the electric breast pump or one of its components if they are damaged.
- Do not place the electric breast pump near appliances that generate a strong electromagnetic field.
- It is advised to consult an approved healthcare professional before using this breast pump if you have an active implanted device that might be affected by a magnetic field (for example, a pacemaker or an implantable cardioverter-defibrillator).
- If expressing causes discomfort or pain, stop the appliance and place your finger between the breast and the nipple shield to separate them. Then, remove the nipple shield from the breast.

- Expressing your milk should not cause discomfort or pain; if you are unable to express your milk after 5 minutes of use, stop your session and try again a little later. **Consult a healthcare professional if you have a doubt.**

#### **Warnings to avoid the transmission of infectious diseases:**

- If you are a mother and infected with hepatitis B, hepatitis C or HIV (human immunodeficiency virus), expressing breast milk will not reduce or eliminate the risk of transmitting the virus to your baby through breast milk.

#### **Precautions when using the electrical unit/lithium battery**

- The lithium battery is built-in and cannot be removed. Do not dismantle.
- Do not expose the electrical unit to direct sunlight or heat.
- Recharge the breast pump once a month when not used regularly.
- Repairs must be carried out by Beaba: contact our customer service.
- Do not leave the electric breast pump unsupervised when connected to a power source.
- Unplug the mains adapter from the wall socket when the breast pump is not in use, unless the battery is charging. Unplug the mains adapter when charging is complete.
- Keep the electrical unit dry and do not immerse it in water or place it under running water.
- Do not pick up an electrical appliance that has fallen into water. Unplug it immediately.
- This breast pump is protected against overheating: when the normal operating temperatures are exceeded, the pump unit triggers an alarm, switches off and stops charging.
- This appliance must only be supplied at safety extra low voltage corresponding to the marking on the appliance.

**THIS APPLIANCE IS NOT A MEDICAL DEVICE.**

#### **INTENDED USE**

The Béaba electric breast pump is designed to express the breast milk of breastfeeding mothers. The breast pump can be used in the event of difficulties encountered with breastfeeding or when mother and baby are separated. The appliance is designed to be used by just one person.

This appliance is not a medical device.

## OVERVIEW OF PRODUCT & ACCESSORIES

x1

x1

x1

x1

21mm x1

- 1 Pump electrical unit with 180 ml collection bottle (with 1 25 mm nipple shield, 1 pump body, 1 silicone membrane (diaphragm), 1 hygiene cover).
- 2 Instructions
- 3 1 bottle lid
- 4 Cotton storage bag
- 5 USB-C cable
- 6 1 21 mm nipple shield

## **FUNCTIONS**

This breast pump offers the following functions and characteristics:

- Milk collection kit with silicone nipple shields for gentle pressure, vacuum and massage to stimulate lactation while expressing milk.
- Easy assembly and cleaning.
- Expressing in three phases that imitates baby's natural suckling.
  - "Stimulation" mode: effectively massages the nipple and areola to stimulate the ejection reflex. 9 adjustable power levels.
  - "Express" mode: imitates baby's suckling rhythm. 9 adjustable power levels.
  - "2-in-1" mode: combines stimulation and expressing, to optimise the milk ejection reflex.
- Portable breast pump: takes up very little space. Works with the built-in rechargeable battery or via USB to guarantee complete flexibility whether at home or when travelling.
- Backflow design: silicone membrane acting as a barrier to prevent the backflow of breast milk.
- Touch screen for simple use.
- Low noise level for a quiet and comfortable experience.
- Rechargeable with a mains adapter. Built-in rechargeable lithium battery.

## **SELECTING THE CORRECT NIPPLE SHIELD SIZE**

- 1) Measure the diameter of the base of your nipple (in mm).
- 2) Add 4 mm to this measurement, this is the size of your nipple shield. A space of about 4 mm between your nipple and the nipple shield allows the nipple to expand while pumping your milk. For example, if the base of your nipple measures 21 mm in diameter, the size of your nipple shield should be 25 mm.

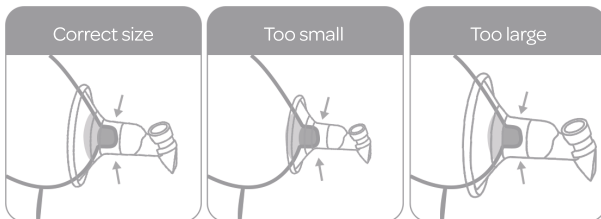
N.B.: your breast pump is supplied with a 25 mm nipple shield and a 21mm nipple shield (which are the most commonly used sizes). However, if necessary, you can order 17 or 29 mm nipple shields from our customer service.

If you are uncertain about the size of the nipple shield you require, print our measurement guide by scanning the QR code below.


How to know whether the nipple shield is the right size?

- Before pumping your milk, it should be possible to insert your nipple with very little or no areola in the tunnel.
- While expressing your milk, your nipple should be centred in the middle and be entered and removed from the tunnel without rubbing, redness or pain.


## PREPARING THE BREAST PUMP

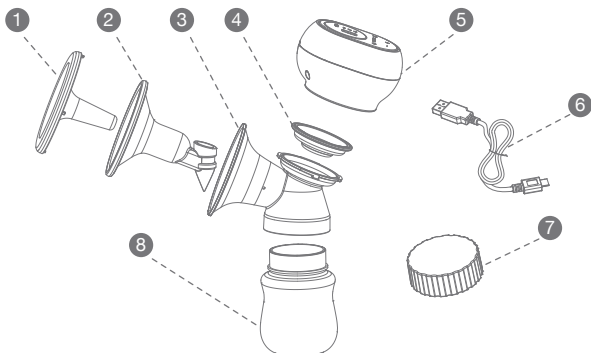- 1 Hygiene cover (PP: polypropylene)
- 2 nipple shield (silicone)
- 3 Pump body (PP: polypropylene)
- 4 Diaphragm (silicone)
- 5 Electrical unit
- 6 USB-C cable
- 7 Bottle lid (PP: polypropylene)
- 8 180 ml bottle (PP: polypropylene)

The following components, which come into direct contact with breast milk, must be cleaned carefully before each use: silicone nipple shield, pump body, diaphragm, collection bottle, bottle lid.

### 1. Assembling the nipple shield and the pump body

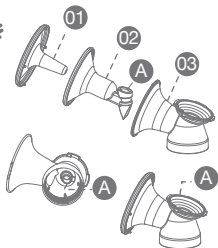
Insert the nipple shield **02** into the pump body **03**. You will obtain the expressing collection kit **A**.



Comment: Remove the hygiene cover before starting to pump your milk.

### 2. Assembling the bottle

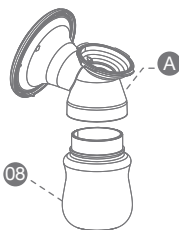Assemble the collection kit **A** on the collection bottle **08**.

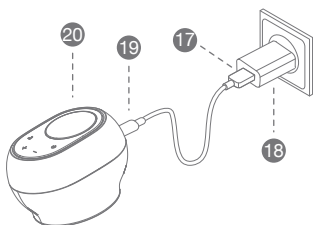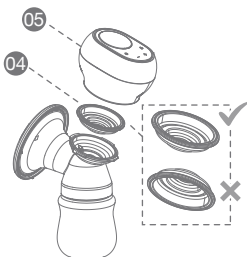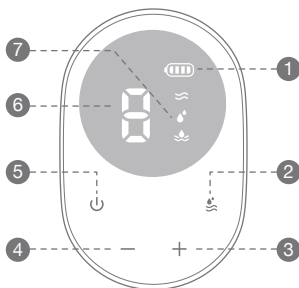
3. Assemble the diaphragm with the electrical unit  
Start by placing the diaphragm 04 on the pump body.  
Place above the electrical unit 05 and tighten it clockwise.


4. Connect the pump unit to the power supply to recharge it  
Insert the cable's USB port 17 into your mains adapter 18.  
Then insert the cable's USB-C 19 connector into the port on the pump unit 20. Connect the output of your mains adapter 18 to the socket.

## DISPLAY SCREEN AND INSTRUCTIONS

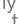1) Before each use: clean well and check the condition of your appliance and the milk collection kit (see following section).

- 2) Press and hold the button  <sup>5</sup> to switch on the appliance. It starts in level 1 stimulation mode by default when using for the first time. The icon  is displayed <sup>7</sup>. The power can be adjusted to 9 levels by pressing the  $-/+$  buttons (<sup>4</sup> and <sup>3</sup>). The screen displays the current power  <sup>6</sup>.
- 3) During operation, press briefly on the  <sup>2</sup> button to switch to express mode. The icon  <sup>7</sup> is displayed. The appliance starts in level 1 stimulation mode by default when using for the first time. The power can be adjusted to 9 levels by pressing the  $-/+$  buttons (<sup>4</sup> and <sup>3</sup>). The screen displays the current power  <sup>6</sup>.
- 4) During operation, press briefly on the  <sup>2</sup> button to switch to 2-in-1 mode. In 2-in-1 mode, the icon  <sup>7</sup> is displayed. The breast pump stimulates for 5 cycles and then expresses for 5 cycles. By default, level 1 is selected when using for the first time. The power can be adjusted to 9 levels by pressing the  $-/+$  buttons (<sup>4</sup> and <sup>3</sup>). The screen displays the current power  <sup>6</sup>.
- 5) During operation, press briefly on  <sup>5</sup> to pause or resume the pumping mode and power level. If the pause time exceeds 5 minutes, the machine will stop automatically and switch to standby.
- 6) After you have finished pumping your milk, press and hold  <sup>5</sup> to switch off. Otherwise, the appliance will stop automatically and enter standby mode if not operated after 30 minutes.
- 7) Then, unscrew the bottles and close them with the lids. The two elements (bottle and lid) are suitable for the fridge and freezer. You can store your expressed milk in the bottle or pour it into a recipient of your choice suitable for storing milk.
- 8) Recommendations for storing your milk in the bottle supplied:

## **WARNING!**

- Always remove the bottle from the motor unit before storing.
- Make sure that the bottle and its lid are not cracked or damaged before each use.»
- The bottle must not be used to reheat the breast milk. Do not use a microwave or bain-marie to reheat milk.
- Do not defrost the milk in the open but in the fridge.
- Wash your hands carefully before touching the bottle and avoid touching the inside.
- Never refreeze defrosted breast milk.
- Always check the temperature of the milk before serving it to baby.

**Recommendations:**

- Remove the bottle from the electrical unit after expressing your milk and close it with the lid supplied.
- Remember to label the bottle with the date and time the milk was expressed and the volume of milk collected.
- Be sure not to fill the bottle to the maximum limit (freezing will increase the volume of milk in the bottle).
- Breast milk can be refrigerated, frozen and defrosted in the BEABA bottle. Please follow the instructions below:

Breast milk	Storage	Shelf life
Fresh	Room temperature (22°C max)	4 to 6 hours
	Fridge (0 to 4°C)	3 to 5 days
	Freezer (freezer compartment of fridge)	2 weeks
	Freezer (-18°C)	6 to 9 months
Defrosted	Room temperature (22°C max)	2 hours
	Fridge (0 to 4°C)	24 hours

- **Do not refreeze after defrosting.**
  - **Please note that these times cannot be combined.**
  - These storage and defrosting rules for breast milk are recommendations only.
  - **Consult a healthcare professional or a breastfeeding specialist for further information.**
- Defrosting milk:
    - Do not use a microwave or bain-marie to defrost breast milk.
    - Defrost the milk gradually in the fridge.
    - In the event of an emergency, you can defrost breast milk by placing the bottle in cold or lukewarm water (max. 37°C) and using immediately.
    - To reheat the milk, pour the milk into a suitable container and refer to the manufacturer's instructions.
- 9) The screen displays the battery level  . Once fully charged, the breast pump can work for about 2 hours. When the battery

level is too low, the icon  flashes and the appliance switches off automatically. Recharge the battery fully when it is low. In operating mode, the pump unit is powered when the mains adapter is connected. When the mains adapter is not connected, the pump unit automatically uses the built-in lithium battery for power.

## CLEANING YOUR BREAST PUMP

### Recommendations:

- **When using for the first time**, dismantle, wash and sterilise all the parts of the breast pump that come into contact with the breast milk. After each use, dismantle and wash all the parts of the breast pump that come into contact with the breasts and the breast milk.
- Dismantle and clean the parts that come into contact with the breasts or breast milk immediately after use to avoid the build-up of milk residue that might favour the growth of bacteria.
- Take care when cleaning the silicone cushion (nipple shield, diaphragm and valve). Your breast pump will not work correctly if they are damaged.
- Always wash your hands before touching the breast pump and its parts. Avoid touching the internal parts of the milk collection kit that come into contact with breast milk.
- The electrical unit must **never** be exposed to or immersed in water or other liquids to clean it. Never wash and never sterilise the pump's motor, this part does not come into contact with breast milk. The electrical unit's motor contains electrical components that must remain dry. A clean, damp cloth can be used to clean the pump's motor. Any other cleaning method is considered to be incorrect use and will cancel the warranty.
- The appliance **MUST NEVER BE USED** without a diaphragm: risk of fluids entering the appliance's electrical part.
- Always check all plastic or silicone parts for wear.
- Replace used or damaged parts immediately because they will affect the operation of your breast pump.

### Washable components:

- Hygiene cover
- Diaphragm
- Nipple shields
- Pump body with backflow valve
- 180 ml bottle
- Bottle lid

Always handle these elements gently when cleaning. Your breast pump will not work correctly if they are damaged.

• **Cleaning the washable components (before using for the first time and immediately after use)**

- 1) Rinse the elements in cold, clear water (about +20°C / +68°F)..
- 2) Wash the elements in lukewarm water (about +30°C / +86°F) with a fragrance-free, liquid washing-up product for babies.
- 3) Rinse in cold, clear water (about +20°C / +68°F) for 10 to 15 seconds.
- 4) Place all the cleaned parts on a paper towel or on drip tray and leave to dry. Avoid using dish cloths because they may contain germs and bacteria that might contaminate the breast milk.
- 5) Make sure that all the parts are completely dry before use. Once all the parts are dry, reassemble the milk collection kit.

OR

- 1) Place the elements on the top rack of the dishwasher.
- 2) Make sure that they are completely dry. If necessary, place all the cleaned parts on a paper towel or on drip tray and leave to dry. Avoid using dish cloths because they may contain germs and bacteria that might contaminate the breast milk.
- 3) Make sure that all the parts are completely dry before use. Once all the parts are dry, reassemble the milk collection kit.

• **Sterilising the washable components before using for the first time**

- 1a) Boil in water for a maximum of five minutes.
  - a. During sterilisation, use a teaspoon of citric acid or a little vinegar in 200 ml of water to descale and degrease the pump's parts. Limescale and grease will affect the vacuum and the transfer of milk to the collection kit.
  - b. Boiling or exposing to heat too often may cause wear of plastic and silicone parts, which will affect the operation of your breast pump.

OR

- 1b) Use steam or cold water sterilisation methods. We do not recommend sterilising in a microwave.
- 2) Place all the cleaned parts on a paper towel or on drip tray and leave to dry. Avoid using dish cloths because they may contain germs and bacteria that might contaminate the breast milk.
- 3) Make sure that all the parts are completely dry before use. Once all the parts are dry, reassemble the milk collection kit.

- **Cleaning the electrical unit:**

Disinfect the electrical unit with disinfectant wipes. Wipe the unit with a clean, damp cloth or paper towel before use. Never place the electrical unit under water and never immerse it. Do not use solvents or abrasive products to clean the electrical unit.

**Scan the QR code below to watch our explanatory video:**

## TROUBLESHOOTING

### *Low suction*

#### CAUSES

- 1) Air leak between the parts (nipple shield, pump body)
- 2) The electrical unit's air circuit is blocked.

#### SOLUTIONS

Check each part, reinstall and clean.

### *No suction*

The valve is not installed, incorrectly assembled, deformed or damaged.

Install or replace the valve. Contact the after-sales service if you require spare parts.

### *Unable to start the appliance*

The mains adapter is not connected to the mains plug or the power socket is not connected to the electrical unit. The battery is depleted.

Reconnect the different parts or charge the lithium battery:

### *The display shows a E1 or E2 alarm.*

E1: circuit open  
E2: short circuit

Please contact your retailer or our Béaba customer service.

### *The display shows an E3 alarm:*

The internal temperature of the electrical unit exceeds the normal operating temperature range (below 0°C or above 60°C).

Temperature too low or too high: place the electrical unit somewhere at a normal temperature. Switch off the electrical unit and wait for the temperature to return to normal before using it.

### THE APPLIANCE DOES NOT SWITCH ON

Check your power socket by plugging in another electrical appliance

Make sure that your mains adapter is connected to the breast pump unit correctly.

If you are using a mains adapter and your breast pump does not switch on, try using the battery instead of the mains adapter.

Make sure the battery is fully charged.

If the unit works on the battery, you may have a faulty mains adapter.

### LOW POWER OR NO POWER

Frequent use or incorrect sterilisation methods can cause ageing of the nipple shield. If the nipple shield is worn, replace it immediately (contact our Béaba customer service).

Make sure that the nipple shield fits snugly over the pump body and is securely fastened.

Make sure that the silicone diaphragm is correctly fastened to the pump body.

Check that all the connections are tightened and there are no air leaks.

If your problems with the Béaba breast pump persist or if you have any other questions, please contact our consumer service.

### TECHNICAL DATA

Recommended mains adapter: Input 100-240V 50/60Hz, Output 5VDC, 2A – 10W

Battery type and capacity: Li-ion 3.7V 2200mAh

Timer: 30 minutes

Noise level:  $\leq 45$  dB

Protection class: class II

Operating conditions: temperature 5-40°C, relative humidity 10%-90%

This symbol, displayed on the product or on its packaging, indicates that this product complies with the European Directive 2012/19/EU and must not be disposed of with household waste. It must be taken to an appropriate collection point for recycling electrical and electronic equipment. By ensuring that this product is disposed of in an appropriate way, you will be helping to prevent potential harmful consequences for the environment and human health. Recycling materials will help to conserve natural resources.

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**BEABA ASIA LTD**